# LOUPES

## INSTRUCTIONS for Use and Care

 **ORASCOPTIC™**

INTENTIONALLY BLANK

# INDEX

---

English.....	4	日本語 (Japanese).....	99
<b>Български</b> (Bulgarian).....	11	한국어 (Korean).....	107
中國 (Chinese).....	19	Latvijas (Latvian).....	115
Ceština (Czech).....	27	Lietuvos (Lithuanian).....	123
Dansk (Danish).....	35	Magyar (Hungarian).....	131
Deutsch (German).....	43	Norsk (Norwegian).....	139
Nederlands (Dutch).....	51	Polski (Polish).....	147
Eesti (Estonian).....	59	Português (Portuguese).....	155
Español (Spanish).....	67	Slovenčina (Slovak).....	163
Français (French).....	75	<b>Slovenščina</b> (Slovenian).....	171
<b>Ελληνικά</b> (Greek).....	83	Suomi (Finnish).....	179
Italiano (Italian).....	91	Svenska (Swedish).....	187

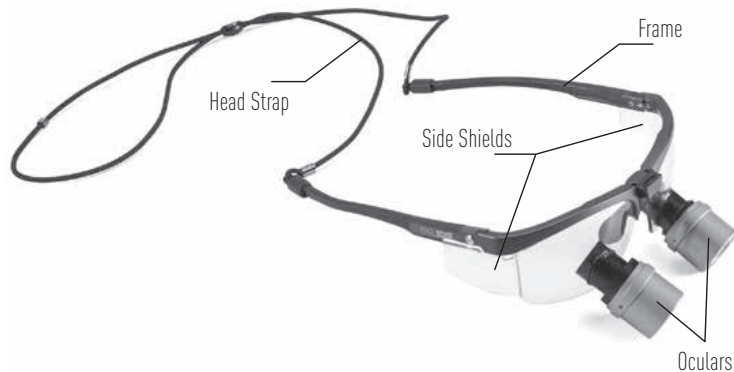
## INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

### Getting Started With Your New Orasoptic Loupe

It is common for new users of magnification to experience some initial disorientation. This may also occur when increasing from a lower magnification power to higher powered loupes. In such cases, a two to three week acclimation period is recommended. If you initially experience dizziness, wear your loupe for no more than one hour at a time and start by performing simple procedures. As your eyes adapt to a magnified field and your hand-eye coordination develops, gradually extend the time you wear your loupe.

Your Orasoptic loupe contains these primary components:

#### Accessories Included

Head Strap, Side Shields, Lens Cleaning Cloth, Optical Screwdriver

Please take special attention during unpacking to verify all components listed above are present. Please call one of our Customer Care Specialists or your authorized Orasoptic distributor to order replacement parts.

#### Indications for Use

Loupes are a magnification device, worn as a visual aid, intended to magnify oral cavities and operating sites. This device is intended to only be used by qualified medical and healthcare professionals including, but not limited to, dentists, dental hygienists, medical doctors and medical professionals.

#### Warnings

1. The device must be used exclusively for the intended purpose described.
2. Do not use the device if any component is damaged, loose, detached, or missing. This includes carrier lenses, telescopes, frame body, temple arms, nose pad, side shields, and flip-up hinge. Parts that are broken, missing, distorted or worn should be replaced immediately. Should such repair or replacement become necessary, ORASOPTIC OR AN AUTHORIZED SERVICE DEALER SHOULD PERFORM ALL REPAIRS TO THIS PRODUCT.

#### Laser Warnings

The human eye is extremely sensitive to laser radiation. Failure to observe the following warnings can result in blindness or other severe and permanent damage to your eyes.

1. **Inspect for damage and fit before each use.** Scratched, cracked, or loose fitting laser loupes or laser inserts may permit the laser beam to reach the eye directly and should not be used.
2. **Ensure the wavelength and power engraved on your laser loupe carrier lenses or laser insert matches the wavelength of the beam being transmitted by your laser.** Different lasers may require a different type of protective eye wear. If the wavelength at which your laser operates is not blocked by your laser loupe or laser insert, you must select different eye wear protection, as your laser loupes or laser insert will not provide protection for your eyes.
3. **Never look directly into a laser beam even when wearing your laser loupe or laser insert.** Your laser loupe is designed to only protect against incidental exposure from stray or diffused reflection of the laser beam's energy.
4. **Your laser loupe or laser insert must be worn at all times while operating your laser.**

**Contraindications** None known.

**Adverse Reactions** None known.

## INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

### Precautions

1. Before use, ensure all screws and removable components are secured to the device.
2. DO NOT
  - Do not disassemble or modify the device
  - Do not spray directly onto the device
  - Do not run any part of the device under water or submerge in any liquid or an ultrasonic cleaner
  - Do not use an autoclave, chemiclave, glutaraldehyde, iodophor, or any other sterilization method
  - Do not use a cleaner with alcohol concentration greater than 70%
  - Do not use paper towel to clean the lenses

### Transport and Storage

### Loupe Prescription Service

If your eye prescription changes, you can send your loupe into our optical laboratory to install your new prescription. When coordinating the return of your loupe, please allow sufficient time for the installation of your prescription.

### Cleaning and Repair Service

Orasoptic offers a loupe cleaning and repair service that is recommended every two years. Optical lab technicians will clean and disinfect your telescopes and frame and replace worn-out screws, nose pad, temple tips, side shields, and head strap.

Orasoptic continuously improves its products as a result of customer feedback. After service, products may reflect improvements in appearance and function.

### Cleaning and Disinfecting

Your loupe is a precision optical instrument manufactured from high quality components. With proper care, it will provide a long and useful life. You must adhere to the following instructions for proper cleaning and disinfecting, or damage may result.

#### DO NOT

- Do not spray directly onto the device
- Do not run any part of the device under water or submerge in any liquid or ultrasonic cleaner
- Do not use an autoclave, chemiclave, glutaraldehyde, iodophor, or any other sterilization method
- Do not use a cleaner with alcohol concentration greater than 70%
- Do not use a paper towel to clean the lenses

### Cleaning

You may use soap and water, to clean all parts of the device. Dampen or spray a soft cloth and then wipe the device. Squeeze excess fluid out of the cloth or wipe before using. Use a lens cleaning cloth with lens cleaning solution to remove any streaks from the lenses. One optical cleaning cloth has been provided with your purchase. Please call one of our Customer Care Specialists or your authorized Orasoptic distributor to purchase additional lens cleaning cloths.

### Disinfecting

You may disinfect the device with a low-alcohol disinfectant. Apply the disinfectant to a soft cloth and then wipe the device. Squeeze excess fluid out of the cloth or wipe before using. Use a lens cleaning cloth with lens cleaning solution to remove any streaks from the lenses. One lens cleaning cloth has been provided with your purchase. Please call one of our Customer Care Specialists or your authorized Orasoptic distributor to purchase additional lens cleaning cloths.

### Recommended Disinfectants

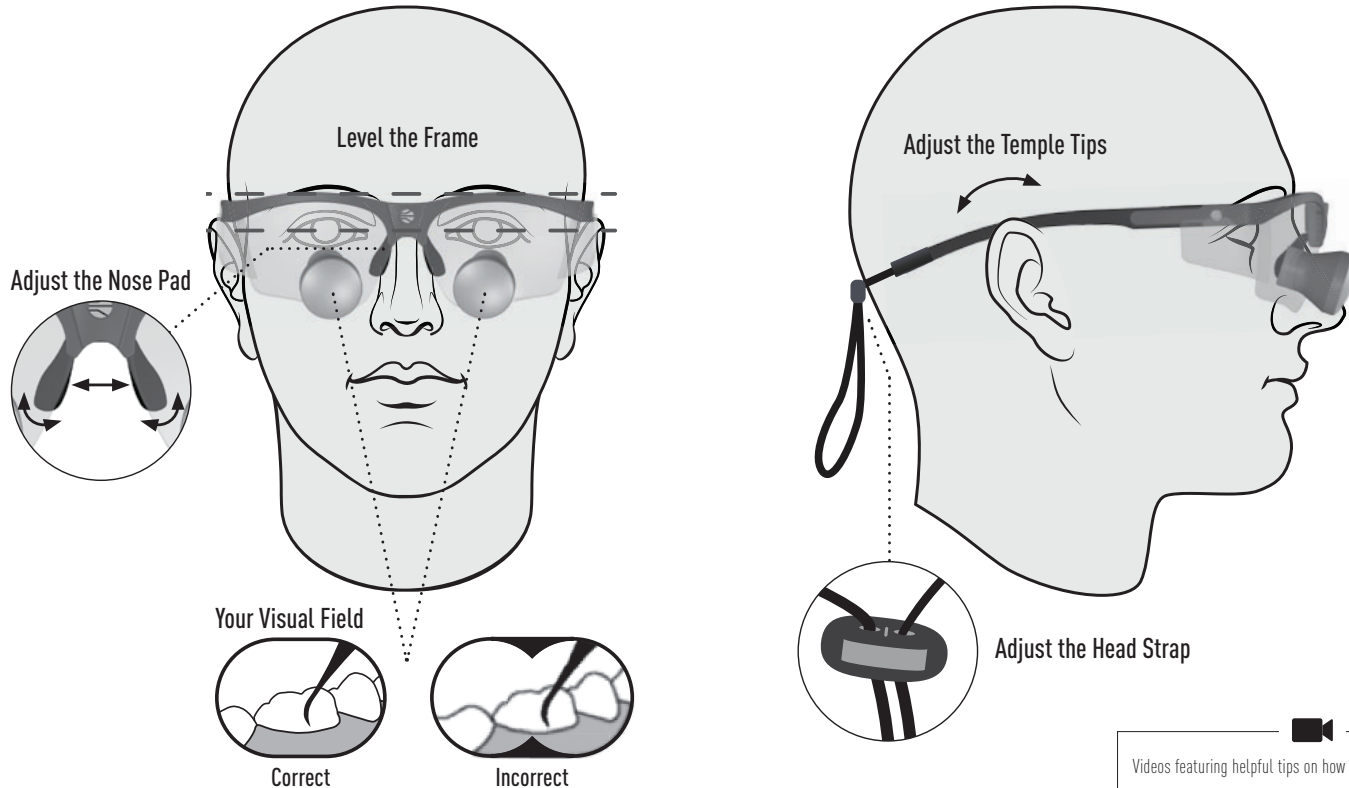
CaviCide™, CaviCide1™, CaviWipes™, or CaviWipes1™

## INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

### Precision Tuning Your Loupe

Although your loupe has been fully tested and customized by our laboratory technicians, they may need to be initially adjusted to your individual facial geometry. Please read through the following guide to learn how to adjust and care for your new loupe.

 Videos featuring helpful tips on how to adjust and clean your loupe are available at [Orasoptic.com/support/videos](https://www.Orasoptic.com/support/videos)

## Precision Tuning Your Loupe

### Level the Frame

Ensuring the frame sits level on your face is crucial. Before any other fine-tuning adjustments can be made, your frame must sit level on your face.

1. Place the frame on your head while in front of a mirror to level the top of the frame with the pupils of your eyes.
2. If the frame cannot be leveled with your pupils, visit a local optician to adjust your frame, or call one of our Technical Support Specialists to assist you with properly adjusting your frame.

**⚠ Attempting to adjust the frame beyond what is defined in these instructions without assistance may result in damage.**

### Adjust the Nose Pad

The nose pad on most frames is adjustable. Adjusting the nose pad will change how the frame rests on your nose and allow you to properly see through the telescopes.

- To lower the frame on your face, spread the nose pad apart.
- To raise the frame on your face, pinch the nose pad together.
- If the frame is not centered on your face, bend one nose pad or the other as needed until the frame is centered.
- Flatten or twist the ends of the nose pad for additional comfort.

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1. Change the declination angle of your telescopes by tilting the flip-up hinge at the lower hinge joint.
2. Fine-tune the declination angle of your telescopes by adjusting the vertical slide depth. Use an optical screwdriver to slightly loosen the mounting screw. Move the vertical slide up for a shallower angle or down for a steeper angle, then tighten the mounting screw to lock the new position. **The mounting screw also secures the mounting clip to your frame. While the mounting screw is loosened, use one hand to keep the mounting clip secured to the frame.**
3. Fine-tune the pupillary distance of your telescopes by rotating the pupillary distance (PD) knob on the hinge to adjust the distance between telescopes to align with your eyes.

### Adjust the Temple Tips and Head Strap

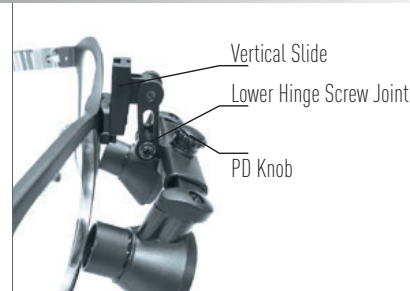
Adjusting the temple tips and head strap are recommended to help distribute weight around your head for additional comfort. The temple tips on most frame styles can be custom contoured to your head shape to optimize comfort.

1. Place the loupe on your head and bend the temple tips to conform to the shape of your head and down behind your ears if desired. The temple tips are designed to maintain your desired adjustments.
2. Tighten the head strap bead behind your head. Your loupe should fit comfortably with minimal pressure on your nose.
3. To take off your loupe, loosen the head strap bead, grasp both temple arms, and pull the frame up and over your head.

**⚠ Removing your loupe by grasping a single temple arm may put additional stress on the frame over time.**

### Your Visual Field

When the focus is properly adjusted, the two circles that form the field of view should appear as an ellipse and be in focus. Due to varying facial geometry, sometimes the preset convergence angle may not be aligned properly for your eyes. If the field of view is not clear, or you are experiencing double vision or eye strain, reposition the frame on your face by adjusting the nose pad, temple tips, and head strap until there is a clear field of view.

## INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

### Adjustable Magnification

#### EyeZoom™ Series Oculars

EyeZoom telescopes can transition between multiple magnification powers.

1. To change the magnification, grasp the black ridged portion of each telescope and turn it so the arrow aligns with your preferred magnification level.
2. Ensure both telescopes are set to the same power.

#### EyeZoom™ Mini

2.5x | 3.5x

#### EyeZoom™

3x | 4x | 5x

#### EyeZoom™ Max

3.5x | 6.5x

#### OmniOptic™ Telescope System

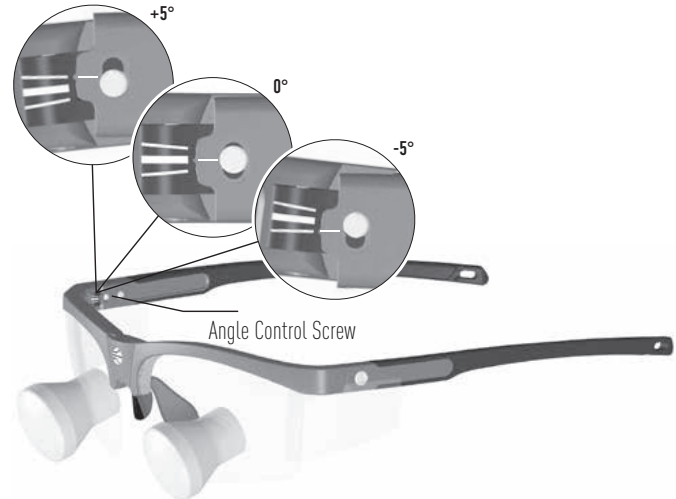
Telescope anchors will be installed in the carrier lenses of your loupe frame.

1. Insert a matching pair of telescopes into the anchors and they will be magnetically secured. Each telescope is assembled with a key-slot to ensure it is properly aligned in the anchor.
2. To remove telescopes from the anchors, hold onto the loupe frame with one hand while pulling each telescope out of its anchor. It will take reasonable force to disengage the magnetic interface.
3. When your loose telescopes are not being used, they should be stored in your loupe case.

### ErgoEdge / RDH Elite Edge Frame

The ErgoEdge model frame employs a unique feature that allows you to add or subtract 5 degrees of declination (tilt) to your telescopes. Three different angle settings are displayed on the inside of each temple arm where the arm interfaces with the front of the frame.

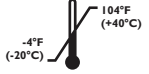
1. From the optical laboratory, the declination of your telescopes will be set to your prescribed angle and aligned to the middle marking on each temple arm.
2. To change declination, loosen the angle control screw on each temple arm and rotate the arms upward or downward to align the arm with a different marking on the frame front, then tighten the angle control screw to lock the temple arm into that position.
  - Top marking: increase declination (steeper angle)
  - Bottom marking: decrease declination (shallower angle)
3. Both temple arms should be set to the same angle or clear vision will be compromised.
4. The nose pad may need to be adjusted after changing declination.

## INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

Symbol Glossary		
 Medical Device	 Engraving Content	 UK Conformity Assessed
 Reference Number / Part Number	 Address of Distributor	 Prescription Device
 Sales Order Number	 Shipping Destination	 Authorized Representative in the European Union
 Purchase Order Number	 Consult Instructions for Use	 Authorized Representative in Switzerland
 Work Order Number	 Address of Manufacturer	 Authorized Representative in United Kingdom
 Serial Number	 Country of Manufacturer	 Temperature Limitations
 Unique Device Identification	 Date of Manufacture	 Humidity Limitations
 Customer Information	 Phone number	 Pressure Limitations
 Order Type	 Email address	
 Use Caution	 European Conformity	

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INTENTIONALLY BLANK

INTENTIONALLY BLANK

## Technical Support/Customer Service: 1.800.369.3698

Outside the U.S. Call Your Local Distributor

© 2020 Metrex Research, LLC dba Orasoptic

Patent Pending

The trademarks noted in these instructions are owned by Metrex Research LLC and/or Kerr Corporation, registered in the U.S. and other countries.

Metrex Research, LLC dba Orasoptic  
6650 Ronald Reagan Ave. | Madison, WI 53704

USA

1.800.369.3698 | 608.831.2555

[Orasoptic.com](http://Orasoptic.com)

**Kerr Italia S.r.l.**, Via Passanti,  
174 Scafati (SA), 84018, Italy

**Nobel Biocare Services AG**, Balz Zimmermann-Strasse 7  
8302 Kloten, Switzerland

**Kerr UK Ltd.**, Orega Stockley Park, 4 Longwalk Road,  
Stockley Park, Uxbridge UB11 1FE United Kingdom

UK  
CA

CE

Rx ONLY

962075-2 Rev. F NOV2022