ZEON™ ENDEAVOUR™ Series

INSTRUCTIONS for Use and Care

ZEON™
ENDEAVOUR™

ZEON™
ENDEAVOUR™
TruCOLOR™

ZEON™
ENDEAVOUR™
XL

ZEON™
ENDEAVOUR™
XL TruCOLOR™

 **ORASCOPTIC™**

INTENTIONALLY BLANK

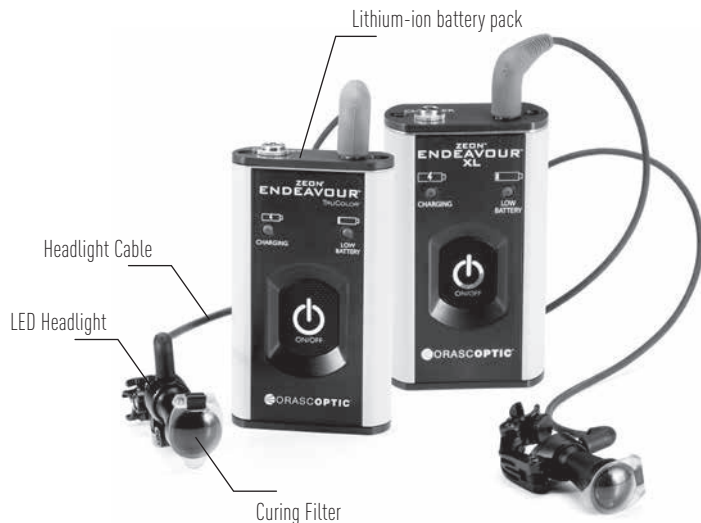
INDEX

English.....	4	日本語 (Japanese).....	99
Български (Bulgarian).....	11	한국어 (Korean).....	107
中國 (Chinese).....	19	Latvijas (Latvian).....	115
Čeština (Czech).....	27	Lietuvos (Lithuanian).....	123
Dansk (Danish).....	35	Magyar (Hungarian).....	131
Deutsch (German).....	43	Norsk (Norwegian).....	139
Nederlands (Dutch).....	51	Polski (Polish).....	147
Eesti (Estonian).....	59	Português (Portuguese).....	155
Español (Spanish).....	67	Slovenčina (Slovak).....	163
Français (French).....	75	Slovenščina (Slovenian).....	171
Ελληνικά (Greek).....	83	Suomi (Finnish).....	179
Italiano (Italian).....	91	Svenska (Swedish).....	187

Getting Started With Your New Zeon Endeavour Series Headlight

This lightweight LED headlight system was designed to attach directly to a majority of existing through-the-lens (TTL) and flip-up loupes. The Zeon Endeavour series battery packs are optimized for use with the headlight that ships with your device. Using other Orasoptic or competitor headlights is not supported.

The Zeon Endeavour series system includes these main components:

Accessories Included

Power Adapter, Spiral Temple Clips, Curing Filter, Optical Screwdriver

Please take special attention during unpacking to verify all components listed above are present. Please call one of our Customer Care Specialists or your authorized Orasoptic distributor to order replacement parts.

Indications for Use

The Zeon Endeavour series is an illumination device attached to spectacles, worn as a visual aid, intended to illuminate oral structures and operating areas. This device is intended to only be used by qualified medical and healthcare professionals including, but not limited to, dentists, dental hygienists, medical doctors and medical professionals.

This device is suitable in professional healthcare facility environments, specifically physician/dental offices, clinics, limited care facilities, freestanding surgical centers, hospitals (emergency rooms, patient rooms, intensive care, surgery rooms except near high frequency surgical equipment, outside the radio frequency shielded room of equipment for magnetic resonance imaging).

Precautions

1. Before use, ensure all screws and removable components are secured to the device.
2. To prolong the life of the batteries, do not store in a fully charged or discharged state.
3. Only use the headlight that ships with the device.
4. DO NOT
 - Do not disassemble or modify the device
 - Do not remove the product serial number labeling
 - Do not begin a procedure while the low-battery indicator light is illuminated or flashing
 - Do not hang or pull on exposed cables
 - Do not tightly bend or wrap the headlight cable
 - Do not spray directly onto the device
 - Do not run any part of the device under water or submerge in any liquid or an ultrasonic cleaner
 - Do not use an autoclave or chemiclave, glutaraldehyde, iodophor, or any other sterilization method
 - Do not use a cleaner with alcohol concentration greater than 70%

Contraindications None known.

Adverse Reactions None known.

Warnings

1. The device must be used exclusively for the intended purpose described.
2. Do not use the device if any component is damaged, loose, detached, or missing. This includes battery packs, headlight, cable, and electrical contacts/plugs. Parts that are broken, missing, distorted or worn should be replaced immediately. Should such repair or replacement become necessary, ORASCOPTIC OR AN AUTHORIZED SERVICE DEALER SHOULD PERFORM ALL REPAIRS TO THIS PRODUCT.
3. The use of power adapters and cables other than those provided for use with the device may result in increased emissions or decreased immunity and may cause the system to be non-compliant with the requirements of IEC 60601-1-2. Only the power adapter and cable that ships with the device should be used to power the Zeon Endeavour series.
4. This product maintains compliance with blue light testing standards for safety set by the International Electrotechnical Commission (IEC). As with all lighting products, there are risks to staring directly into the light.
 - IEC 62471 Risk Group 2 - CAUTION: Possibly hazardous optical radiation is emitted from this product. Do not stare at operating lamp. May be harmful to the eyes.
5. Misuse of the battery may cause them to become hot, explode, or ignite and may cause serious injury. The following safety warnings must always be followed:
 - Never open or modify the battery pack or power adapter.
 - Do not use outside.
 - Protect from fluids and damp environments.
 - Maintain operational environment of 32° to 104°F (0° to 40°C).
 - Do not short circuit the electrical contacts.
 - Do not bring any part of the device into contact with fire or severe heat over 158°F (70°C).
 - If you determine that the device is hot, emits an odor, or changes color or shape, disconnect power immediately. The device must not be used again.
6. Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
7. Portable radio frequency communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 12 inches (30 cm) to any part of this device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Cleaning and Disinfecting

The Zeon Endeavour series is a precision optical-electrical instrument manufactured from high quality components. With proper care, it will provide a long and useful life. You must adhere to the following instructions for proper cleaning and disinfecting, or damage may result.

DO NOT

- Do not spray directly onto the device
- Do not run any part of the device under water or submerge in any liquid or ultrasonic cleaner
- Do not hang or pull on exposed cables
- Do not use an autoclave, chemiclave, glutaraldehyde, iodophor, or any other sterilization method
- Do not use a cleaner with alcohol concentration greater than 70%

Cleaning

You may use soap and water to clean all parts of the device. Dampen or spray a soft cloth and then wipe the device. Squeeze excess fluid out of the cloth or wipe before using. If your eyewear is heavily soiled, first remove the headlight and then clean your eyewear. Dry your eyewear thoroughly before reinstalling the headlight.

Disinfecting

You may disinfect the device with a low-alcohol disinfectant. Apply the disinfectant to a soft cloth and then wipe the device. Squeeze excess fluid out of the cloth or wipe before using.

Recommended Disinfectants

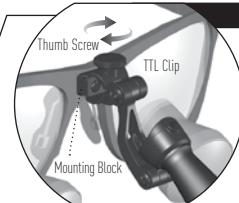
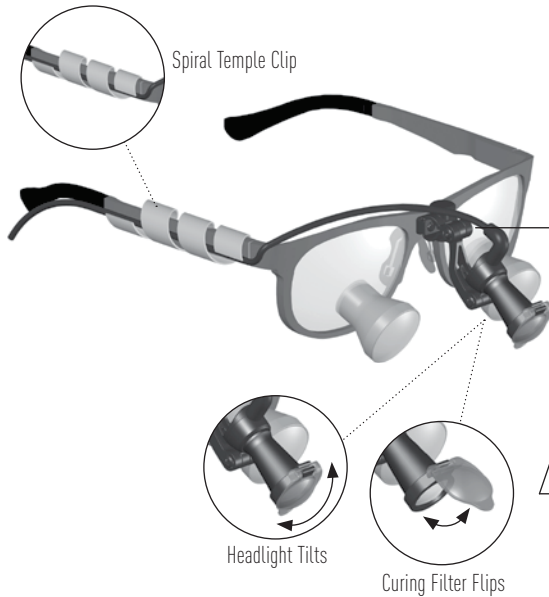
CaviCide™, CaviCide1™, CaviWipes™, or CaviWipes1™

Transport and Storage


Connecting Your Zeon Endeavour Series to a Loupe

Secure the headlight cable to the temple arm of your frame by wrapping the spiral temple clip around both the temple arm and cable. The cable also includes an adjustable gown clip to keep it out of the working area.


Through-the-lens (TTL) Mount

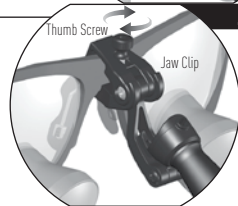Slide the clip onto the mounting block and tighten the thumb screw on top. If a mounting block is not attached to the bridge of your loupe frame, attach one using the parts and instructions provided with your loupe connection kit.


Jaw Mount

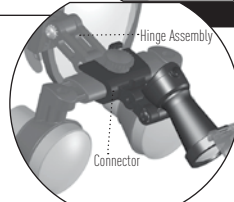Secure to the bridge of your frame by loosening the thumb screw on top to open the jaw arms, then tighten the thumb screw to close the jaw arms over the top and bottom of your frame bridge.

Flip-up Mount

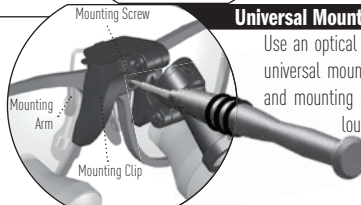Secure to the hinge assembly by pushing the connector onto the front of the hinge. Orasoptic offers different flip-up clips to secure to various Orasoptic and competitor flip-up hinges. Even if your flip-up clip looks different from what is shown, the method to secure is still the same.

Universal Mount

Use an optical screwdriver to loosen the mounting screw and open the mounting arms. Push the universal mount down over the top of your frame with mounting clip in front of the carrier lens and mounting arms behind the carrier lens. Center the universal mount over the bridge of your loupe frame and tighten the mounting screw until the device is secure.

Operating Your Zeon Endeavour Series

Charging the Battery Pack

The Zeon Endeavour series battery pack is partially charged when shipped and should be fully charged prior to use. To charge the battery pack, plug the power adapter into an electrical outlet and into the charger connection port of the battery pack. **The battery pack is capable of powering the headlight while charging.**

- The low battery indicator light will illuminate **SCARLET** when the battery pack is nearly depleted and needs to be charged.
- The low battery indicator will blink **SCARLET** when the battery pack is too low to operate the headlight.
- The charging indicator light will illuminate **AMBER** while the battery pack is charging.
- The charging indicator light will turn off when the battery pack is fully charged.

Operating the Battery Pack

The Endeavour series uses capacitive touch controls, making the light simple to operate during clinical procedures. Touch the control pad with any part of your hand to turn the system on and off, or to change the brightness of the headlight.

1. Plug the headlight cable into the top of the headlight and into the headlight connection port of the battery pack.
2. Touch and hold the control pad for a minimum of 2 seconds to turn the headlight on or off.
3. To increase the brightness, tap the control pad. With each tap, the brightness increases until the highest setting is reached. The light returns to the lowest setting by tapping the control pad again.
4. Place your loupe and headlight on your head and adjust to a comfortable working position. Turn the headlight on and tilt it to the appropriate angle to illuminate your visual field while looking through the telescopes.

CAUTION

Point the headlight away from your eyes
because the light beam is strong.

Operating Your Zeon Endeavour Series

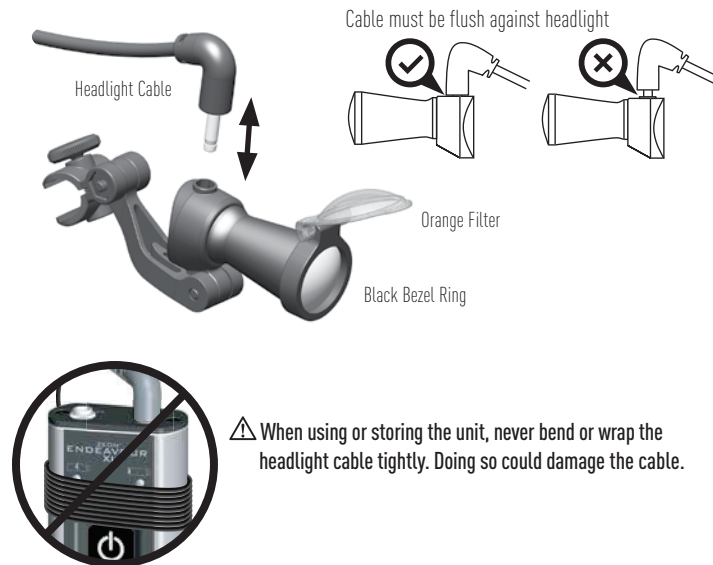
Connecting the Cable to the Headlight

The Zeon Endeavour series cable attaches to the top of the headlight. The cable can easily detach from the headlight, allowing the user to uncouple their loupe from the cable and battery pack. **When connecting the cable to the headlight, it is important to push the cable down firmly until it is completely flush against the headlight.** This will reduce any chance of fluid ingress while also ensuring a stable electrical connection. If you experience issues with the headlight flickering on and off, or not turning on at all, ensure the cable has not come loose.

Using the Curing Filter

The curing filter should be used when working with light-cured composites. When applying these materials, unfiltered light from the headlight can cause premature curing of the composite. The curing filter is made up of two components; the black bezel ring that is secured to the end of the headlight and the orange filter that can be flipped up and down as needed during procedures.

- If you don't require the curing filter for your practice, you can remove it from the headlight. To remove the curing filter, flip the orange filter to the UP-position and carefully pull the black bezel ring off the headlight. **Do not pull on the orange curing filter itself.**
- To attach the curing filter, flip the orange filter to the UP-position and carefully push the black bezel ring onto the headlight, making sure the hinge is at the top. **Do not push on the orange curing filter itself.**

Troubleshooting

Problem	Solution
Headlight flickers on and off or does not turn on at all	Make sure the headlight cable is properly connected to the battery pack and to the headlight. The headlight cable must be completely flush against the headlight to maintain a stable electrical connection.
Low battery indicator light is illuminated scarlet	Indicates the battery is low while the headlight is turned on. Charge the battery.
Low battery indicator light is blinking scarlet	Indicates the battery is low while the headlight is turned off. Charge the battery.
Charging indicator light does not illuminate amber during charging	Make sure the battery pack is connected to the correct power supply and the power supply is plugged into an outlet.
Charging indicator light is blinking amber	Indicates a charging fault. Disconnect and then reconnect the power supply cable from the battery pack.

INSTRUCTIONS FOR USE AND CARE

Orascoptic™ Zeon Endeavour™ Series

Electromagnetic Compatibility Test	
CISPR 11 - Emissions: Radiated RF	Group 1, Class A
IEC 61000-3-2 - Emissions: Harmonic	Class A
IEC 61000-3-3 - Emissions: Voltage Fluctuations / Flicker	Complies
IEC 61000-4-2 - Immunity: Electrostatic Discharge	+8 kV contact ±15 kV air
IEC 61000-4-4 - Immunity: Electrical Fast Transient / Burst	+2 kV for power supply lines ±1 kV for input/output lines
IEC 61000-4-5 - Immunity: Surge	+1 kV differential ±2 kV common
IEC 61000-4-11 - Immunity: Voltage Dips and Short Interruptions	<5 % UT for 0.5 cycles 40 % UT for 5 cycles 70 % UT for 25 cycles <5 % UT for 5 seconds
IEC 61000-4-8 - Immunity: Power Frequency Magnetic Fields	30 A/m, 50/60 Hz
IEC 61000-4-6 - Immunity: Conducted RF	3 Vrms, 150 kHz – 80 MHz, 80 % 1 kHz AM 6 Vrms, ISM and Amateur Radio bands between 150 kHz – 80 MHz, 80 % 1 kHz AM
IEC 61000-4-3 - Immunity: Radiated RF	3 V/m, 80 MHz – 2.7 GHz, 80 % 1 kHz AM
IEC 61000-4-3 - Immunity: Proximity Fields from RF Wireless Communications Equipment	IEC 60601-1-2:2014, section 8.10

The EMISSIONS characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.

Specifications		
	Zeon Endeavour Zeon Endeavour TruColor	Zeon Endeavour XL Zeon Endeavour TruColor XL
Weight of battery pack	4.9 ounces (138 grams)	7.06 ounces (200 grams)
Dimensions of battery pack	Height = 3.6 inches (92 mm) Width = 2.0 inches (52 mm) Depth = 0.8 inches (20 mm)	Height = 3.6 inches (92 mm) Width = 2.1 inches (55 mm) Depth = 1.1 inches (28 mm)
Battery type	Li-Polymer, 3.6 VDC (3120 mAh)	Li-Polymer, 3.63 VDC (6500 mAh)
Light intensity modes	Level 1 = 34 lumens Level 2 = 53 lumens Level 3 = 68 lumens	Level 1 = 53 lumens Level 2 = 68 lumens Level 3 = 85 lumens
Battery life	Level 1 = 18 hours Level 2 = 11 hours Level 3 = 7 hours	Level 1 = 24 hours Level 2 = 17 hours Level 3 = 10 hours
Charge time (each battery pack)	3 hours	6 hours
Color Rendering Index (CRI)	Standard = 70+ TruColor = 90+	
Correlated Color Temperature (CCT)	Standard = 6500K TruColor = 5700K	
Weight of the headlight	.25 ounces (7 grams) With curing filter = .28 ounces (8 grams)	
Diameter of headlight	.55 inch (14 mm)	
Electrical	☑ Class II and internally powered	
Operation mode	Continuous	
Standards	EMC = IEC60601-1-2, class B, EN55011 Safety = IEC60601-1, IEC 62471	
Operating environment	32° to 104°F (0° to 40°C) 10% to 85% relative humidity (non-condensing) 0.5 atm to 1.0 atm (500 hPa to 1060 hPa) atmospheric pressure	

Disposal Instructions

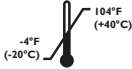
When you plan to dispose of the device, please return to Orascoptic for recycling. The European Union Waste Electrical and Electronic Equipment (WEEE) directive forbids the disposal of electrical and electronic equipment as unsorted municipal waste, requiring that it be collected and recycled or disposed of separately. Hazardous substances are present in electrical and electronic equipment and present potential risk to human health and environment if disposed of in municipal landfills which are not designed to prevent migration of substances into the soil and groundwater. If you need assistance when disposing the device, please call one of our Customer Care Specialists or your authorized Orascoptic distributor.

This device contains lithium-ion cells. Recycle or dispose of the batteries accordingly to national, state, and local regulations.

INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Zeon Endeavour™ Series

Symbol Glossary				
 MD	Medical Device	 Address of Manufacturer	 Rx ONLY	Prescription Device
 REF	Reference Number / Part Number	 Country of Manufacturer	 EC REP	Authorized Representative in the European Union
 SN	Serial Number	 Date of Manufacture	 CH REP	Authorized Representative in Switzerland
 LOT	Lot Number	 Phone number	 UK RP	Authorized Representative in United Kingdom
 UDI	Unique Device Identification	 Email address	 -4°F (-20°C) 104°F (+40°C)	Temperature Limitations
 i	Consult Instructions for Use	 Class II Equipment	 10% 85%	Humidity Limitations
 WEEE	Waste Electrical and Electronic Equipment	 CSA C US 169378 Canadian Standards Association	 0.5 atm (500 hPa) 1.0 atm (1060 hPa)	Pressure Limitations
 Address of Distributor	Address of Distributor	 CE European Conformity		
 Use Caution	Use Caution	 UK CA UK Conformity Assessed		

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INTENTIONALLY BLANK

Technical Support/Customer Service: 1.800.369.3698

Outside the U.S. Call Your Local Distributor

© 2020 Metrex Research, LLC dba Orasoptic

Patent Pending

The trademarks noted in these instructions are owned by Metrex Research LLC and/or Kerr Corporation, registered in the U.S. and other countries.

Manufactured by:
Metrex Research, LLC dba Orasoptic
6650 Ronald Reagan Ave. | Madison, WI 53704

USA

1.800.369.3698 | 608.831.2555

Orasoptic.com

Kerr Italia S.r.l., Via Passanti,
332 Scafati (SA), 84018, Italy

Nobel Biocare Services AG, Balz Zimmermann-Strasse 7
8302 Kloten, Switzerland

Kerr UK Ltd., Orega Stockley Park, 4 Longwalk Road,
Stockley Park, Uxbridge UB11 1FE United Kingdom

UK
CA

CE

Rx ONLY

962071-2 Rev. L Jan2022