ORASCOPTIC™

INSTRUCTIONS

for Use and Care

North America

INTENTIONALLY BLANK

INDEX

English	4
Français (Canada)	13

INSTRUCTIONS FOR USE AND CARE

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Indications for Use

Loupes are a magnification device, worn as a visual aid, intended to magnify oral cavities and operating sites. This device is intended to be used by qualified medical and healthcare professionals including, but not limited to, dentists, dental hygienists, medical doctors, and medical professionals.

Precautions

1. Before each use, ensure all screws and removable components are secured to the device.
2. DO NOT
 - ... spray directly onto the device
 - ... run any part of the device under water or submerge in any liquid or an ultrasonic cleaner
 - ... use an autoclave, chemiclave, glutaraldehyde, iodophor, or any other sterilization method
 - ... use a cleaner with alcohol concentration greater than 70%
 - ... use paper towel to clean the lenses

Contraindications None known.

Adverse Reactions None known.

Transport and Storage

Warnings

1. The device must be used exclusively for the intended purpose described.
2. Do not use the device if any component is damaged, loose, detached, or missing. This includes lenses, telescopes, frame body, temple arms, nose pad, side shields, and flip-up hinge. Parts that are broken, missing, distorted or worn should be replaced immediately. Should such repair or replacement become necessary, ORASCOPTIC OR AN AUTHORIZED SERVICE DEALER SHOULD PERFORM ALL REPAIRS TO THIS PRODUCT.

Laser Warnings

The human eye is extremely sensitive to laser radiation. Failure to observe the following warnings can result in blindness or other severe and permanent damage to your eyes.

1. **Inspect for damage and fit before each use.** Scratched, cracked, or loose fitting laser loupes or laser inserts may permit the laser beam to reach the eye directly and should not be used.
2. **Ensure the wavelength and power engraved on your laser loupe carrier lenses or laser insert matches the wavelength of the beam being transmitted by your laser.** Different lasers may require a different type of protective eye wear. If the wavelength at which your laser operates is not blocked by your laser loupe or laser insert, you must select different eye wear protection, as your laser loupes or laser insert will not provide protection for your eyes.
3. **Never look directly into a laser beam even when wearing your laser loupe or laser insert.** Your laser loupe is designed to only protect against incidental exposure from stray or diffused reflection of the laser beam's energy.
4. **Your laser loupe or laser insert must be worn at all times while operating your laser.**

INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

Cleaning and Disinfecting

Your loupe is a precision optical instrument manufactured from high quality components. With proper care, it will provide a long and useful life. You must adhere to the following cleaning and disinfecting instructions or damage may result. Additionally, it is recommended to follow all instructions provided with your chosen cleaning/disinfecting solution.

Recommended Disinfectants

CaviCide™, CaviCide1™, CaviWipes™, or CaviWipes1™

1. Dampen or spray a soft cloth with your chosen cleaning/disinfecting solution, or use a disinfectant wipe. Squeeze excess fluid out of the cloth or wipe before using.
2. Hold the device securely while cleaning all surfaces with the damp cloth or wipe, including lenses, telescopes, temple arms, and side shields.
3. Discard dirty cloths or wipes and repeat the steps above until the device is clean.
4. If using a disinfectant, allow all surfaces to remain saturated for the proper duration (following instructions provided with your chosen disinfecting solution).
5. Use a clean and dry soft cloth to wipe all remaining cleaning/disinfecting solution from the surface of your device.
6. Dampen a lens cleaning cloth with lens cleaning solution and wipe the inside and outside of the lenses, telescopes, and side shields to remove any streaks left from the cleaning and disinfecting process. One lens cleaning cloth has been provided with your loupe. Please call one of our Customer Care Specialists or your authorized Orasoptic distributor to purchase additional lens cleaning cloths.

DO NOT

- ... spray directly onto the device
- ... use paper towel to clean the lenses
- ... use a cleaning/disinfecting solution with alcohol concentration greater than 70%
- ... run any part of the device under water or submerge in any liquid or ultrasonic cleaner
- ... use an autoclave, chemiclave, glutaraldehyde, iodophor, or any other sterilization method

INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

Getting Started With Your Orasoptic Loupe

Please take a moment to read through these instructions to familiarize yourself with the features of this device. It is common for new users of magnification to experience some initial disorientation. This may also occur when increasing magnification from a lower power. In such cases, a two to three week acclimation period is recommended. If you initially experience dizziness, wear your loupe for no more than one hour at a time and start by performing simple procedures. As your eyes adapt to a magnified field and your hand-eye coordination develops, gradually extend the time worn.

Accessories Included

Head Strap, Side Shields, Customized Storage Case, Optical Screwdriver, Lens Cleaning Cloth, Anti-fog Cloth

Please call one of our Customer Care Specialists or your authorized Orasoptic distributor to order additional or replacement parts.

INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

Loupe Accessories

Orasoptic loupes include accessory items that can be removed if not needed, or replaced if damaged. Please read through the following guide to learn about these accessories.

Side Shields

The side shields offer additional protection. We recommend keeping the side shields installed to maintain the device's ANSI Z87.1 certified protection, but can be removed if preferred.

Head Strap

We recommend tightening the head strap during use to help distribute the weight around your head instead of on your nose. The frame should fit comfortably with minimal pressure on your nose.

Lens Cleaning Cloth

Orasoptic lens cleaning cloths are made of high quality microfiber material to absorb and remove dust, smudges, fingerprints, or other liquid splatter. They are ideal for cleaning lenses, telescopes, and side shields without leaving streaks or scuff marks. They can be used dry or with your chosen cleaning solution as described in the **Cleaning and Disinfecting** section of these instructions.

Anti-Fog Cloth

Orasoptic anti-fog cloths can be used to wipe the telescope lenses prior to use as an additional step to prevent fogging. When using anti-fog cloths, follow instructions provided with the cloth.

One lens cleaning cloth and one anti-fog cloth has been provided with your loupe. Please call our Customer Care Specialists or your authorized Orasoptic distributor to purchase additional cloths.

Services

Loupe Prescription Service

If your eye prescription changes, you can send your loupe into our optical laboratory to install your new prescription.

Cleaning and Repair Service

Orasoptic offers a cleaning and repair service that is recommended every two years. Optical lab technicians will clean and disinfect your telescopes and frame, and replace worn-out screws, nose pad, side shields, and head strap.

Orasoptic continuously improves its products as a result of customer feedback. After service, products may reflect improvements in appearance and function.

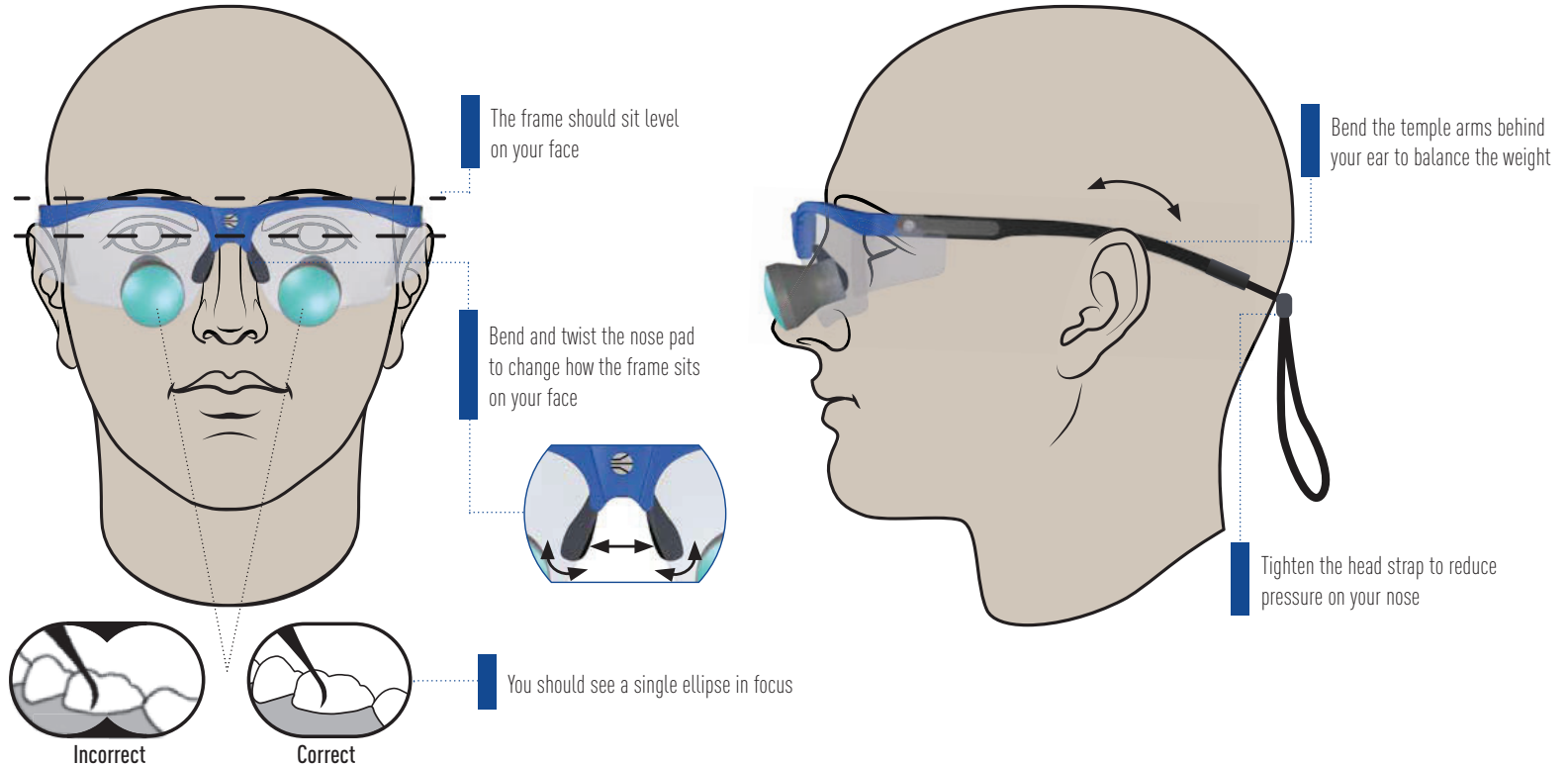
INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

Precision Tuning Your Orasoptic Loupe

Although your loupe has been fully tested and customized by our laboratory technicians, it may need to be initially adjusted to your individual facial geometry. Please read through the following guide to learn how to adjust this device.

INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

Leveling the Frame

It is important that the frame sits level on your face so your visual field is clear and in focus.

1. Place the frame on your head while in front of a mirror to level the top of the frame with the pupils of your eyes.
2. If the frame is not level with your pupils, reposition the frame on your face by adjusting the nose pad and temple arms until the frame is level.
3. If the frame cannot be leveled with your pupils, call one of our Technical Support Specialists to assist you with properly adjusting your frame.

⚠ Attempting to adjust the frame beyond what is defined in these instructions without assistance may result in damage.

Adjusting the Nose Pad

Adjusting the nose pad will change how the frame rests on your nose and allow you to properly see through the telescopes.

- To lower the frame on your face, spread the nose pad apart.
- To raise the frame on your face, pinch the nose pad together.
- If the frame is not centered on your face, bend one side of the nose pad or the other as needed until the frame is centered.
- Flatten or twist the ends of the nose pad for additional comfort.

Adjusting the Temple Arms

We recommend bending the temple arms to conform with the shape of your face and head, allowing the frame to fit more comfortably. While wearing your loupe, bend the temple arms down behind your ears to create a secure fit. The temple arms will maintain your desired adjustments.

Your Visual Field

When the focus is properly adjusted, the two circles that form the field of view should appear as an ellipse and be in focus. Due to varying facial geometry, sometimes the preset convergence angle may not be aligned properly for your eyes. If your field of view is not clear, or you are experiencing double vision or eye strain, reposition the frame on your face by adjusting the nose pad and temple arms until you have a clear field of view.

Videos featuring helpful tips to adjust and clean Orasoptic products are available at [Orasoptic.com/support/videos](https://www.orasoptic.com/support/videos)

INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

Variable Magnification

EyeZoom™ Ocular Series

Orasoptic EyeZoom oculars can transition between multiple magnification powers.

1. To change the magnification, grasp the black ridged portion of each telescope and turn it so the arrow aligns with your preferred magnification level.
2. Ensure both telescopes are set to the same power.

EyeZoom™ Mini

2.5x | 3.5x

EyeZoom™

3x | 4x | 5x

EyeZoom™ Max

3.5x | 6.5x

OmniOptic™ Ocular System

Orasoptic EyeZoom telescopes can transition between multiple magnification powers.

1. Insert a matching pair of telescopes into the anchors and they will be magnetically secured. Each telescope is assembled with a key-slot to ensure it is properly aligned in the anchor.
2. To change the magnification, grasp the black ridged portion of each telescope and turn it so the arrow aligns with your preferred magnification level.
3. Ensure both telescopes are set to the same power.

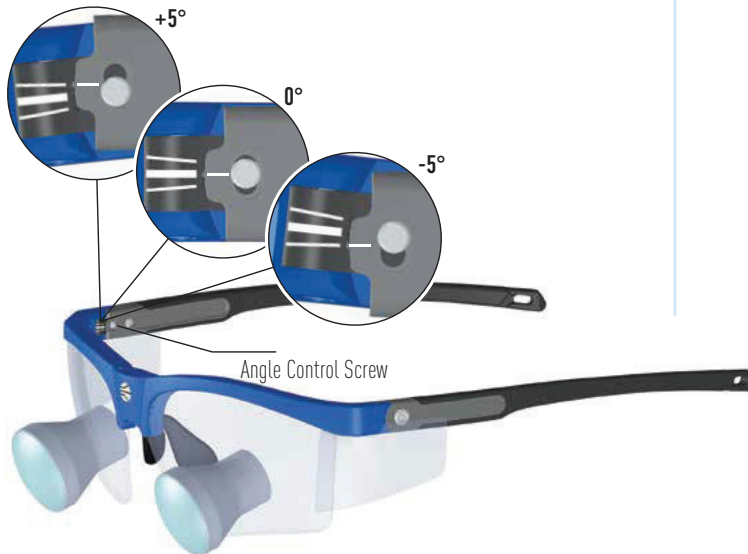
INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

ErgoEdge and RDH EliteEdge Frames

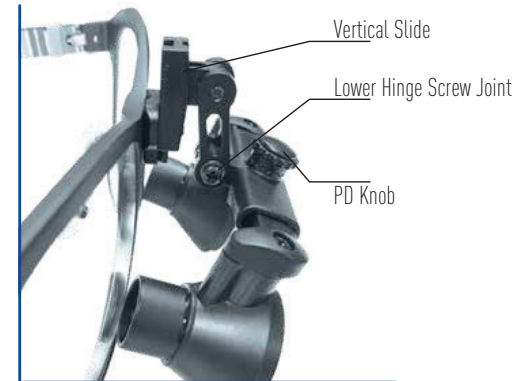
The ErgoEdge model frame employs a unique feature that allows you to add or subtract 5 degrees of declination (tilt) to your telescopes. Three different angle settings are displayed on the inside of each temple arm where the arm interfaces with the front of the frame.

1. From the optical laboratory, the declination of your telescopes will be set to your prescribed angle and aligned to the middle marking on each temple arm.
2. To change declination, loosen the angle control screw on each temple arm and rotate the arms upward or downward to align the arm with a different marking on the frame front, then tighten the angle control screw to lock the temple arm into that position.
 - Top marking: increase declination by 5 degrees (steeper angle)
 - Bottom marking: decrease declination by 5 degrees (shallower angle)
3. Both temple arms should be set to the same angle or clear vision will be compromised.

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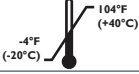
1. Change the declination angle of your telescopes by tilting the flip-up hinge at the lower hinge joint.
2. Fine-tune the declination angle of your telescopes by adjusting the vertical slide depth. Use an optical screwdriver to slightly loosen the mounting screw. Move the vertical slide up for a shallower angle or down for a steeper angle, then tighten the mounting screw to lock the new position. **The mounting screw also secures the mounting clip to your frame. While the mounting screw is loosened, use one hand to keep the mounting clip secured to the frame.**
3. Fine-tune the pupillary distance of your telescopes by rotating the pupillary distance (PD) knob on the hinge to adjust the distance between telescopes to align with your eyes.

INSTRUCTIONS FOR USE AND CARE

Orasoptic™ Loupes

Symbol Glossary		
 Medical Device	 Order Type	 European Conformity
 Reference Number / Part Number	 Phone Number	 UK Conformity Assessed
 Sales Order Number	 Email Address	 Temperature Limitations -4°F (-20°C) to 104°F (+40°C)
 Purchase Order Number	 Website Address	 Humidity Limitations 10% to 85%
 Work Order Number	 Engraving Content	 Pressure Limitations 0.5 atm (500 hPa) to 1.0 atm (1060 hPa)
 Serial Number	 Shipping Destination	 Prescription Device
 Unique Device Identification	 Customer Information	 Authorized Representative for European Union
 Address of Manufacturer	 Consult Instructions for Use	 Authorized Representative in Switzerland
 Country of Manufacturer	 Use Caution	 Authorized Representative in United Kingdom
 Date of Manufacture	 Address of Distributor	

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1.800.369.3698 x2 | 1.608.831.2555 x2

TechSupport@orascoptic.com

Metrex Research, LLC dba Orascoptic
6650 Ronald Reagan Ave. | Madison, WI 53704

USA

1.800.369.3698 | 1.608.831.2555

Orascoptic.com

Rx ONLY

Kerr Italia S.r.l., Via Passanti,
174 Scafati (SA), 84018, Italy

Nobel Biocare Services AG, Balz Zimmermann-Strasse 7
8302 Kloten, Switzerland

Kerr UK Ltd., Orega Stockley Park, 4 Longwalk Road,
Stockley Park, Uxbridge UB11 1FE, United Kingdom

ETL CLASSIFIED

Intertek
5024279

Conforms to AAMI Std ES60601-1 and IEC Std 60601-1-6
Certified to CSA STD C22.2# 60601-1

The trademarks noted in these instructions are owned by Metrex Research, LLC and/or Kerr Corporation, registered in the U.S. and other countries.

© 2022 Metrex Research, LLC dba Orascoptic | Patents: #US 11,250,480, #US 10,107,483

962075-1 Rev. D, NOV2022